For the Quantitative Measurement of Total Testosterone in Human Serum

The concentration of Testosterone in a given specimen determined with assays from different manufacturers can vary due to differences in assay methods and reagent specificity. The results reported by the laboratory to the physician must include the identity of the Testosterone assay method used. Values obtained with different assay methods should not be used interchangeably.

CAUTION: United States Federal law restricts this device to sale and distribution by or on the order of a physician, or to a clinical laboratory; and use is restricted to, by or on the order of a physician.

INTENDED USE

The FastPack® IP and FastPack® Testo Immunoassay is a paramagnetic particle immunoassay for the in vitro quantitative determination of total testosterone in human serum. The FastPack® IP and FastPack® Testo Immunoassay is designed for use with the FastPack® IP System.

SUMMARY

Testosterone (17 β -hydroxyandrost-4-en-3-one) is a C₁₉ steroid hormone and is the major androgen in males. It is predominantly synthesized in the interstitial Leydig cells of the testis and is regulated by interstitial cell stimulating hormone or luteinizing hormone of the anterior pituitary¹. Testosterone is responsible for the masculinization of the genital tract and the development, and maintenance of male secondary sex characteristics, for example, the accessory sex organs, the prostate, seminal vesicles and growth of facial and pubic hair². Testosterone in females is primarily from three sources: the ovaries, the adrenal gland and the peripheral metabolism of the precursor, androstenedione.

Determination of testosterone levels is an integral part in the assessment of hypogonadal states. Some of the major causes of low testosterone levels in males are hypogonadotropic hypogonadism, testicular failure, hyperprolactinemia, Klinefelter's syndrome, hypopituitarism, estrogen therapy and liver cirrhosis. Elevated levels occur in complete androgen resistance (testicular feminization). In females, some of the major causes of increased testosterone levels are ovarian tumors, polycystic ovaries, adrenal tumors and adrenal hyperplasia.⁷

The FastPack® IP and FastPack® Testosterone assay is based on a competitive immunoassay principle: testosterone in the sample (control or patient) competes with testosterone covalently coupled to a solid phase paramagnetic particle for an alkaline phosphatase labeled anti-testosterone monoclonal antibody. The amount of labeled-anti-testosterone antibody bound to the magnetic particle is inversely proportional to the concentration of total testosterone in the sample.

TEST PRINCIPAL

The FastPack® IP and FastPack® Testo Immunoassay is a competitive chemiluminescence assay.

- Primary incubation: Antibody solution (a buffer solution containing a monoclonal C3-testosterone-specific antibody labeled with alkaline phosphatase) [100 μ L] reacts with testosterone from the patients sample, control, or calibrator [100 μ L].
- Secondary incubation: The reaction mixture is added to paramagnetic particle with covalently coupled testosterone. During this incubation, the testosterone-coated beads compete with the sample testosterone.
- Removal of unbound materials: The paramagnetic particles are washed with wash buffer [0.2 mL/wash] to remove unbound materials.
- Substrate addition and detection: Chemiluminogenic substrate [140 μ L] is added to the solid-phase bound complex and results in "glow" chemiluminescence, which is measured using the FastPack® IP Analyzer.
- The amount of bound labeled-antibody is inversely proportional to the concentration of testosterone in the sample.

REAGENTS - Content and Concentration

Each FastPack® IP carton contains:

- 30 FastPack® IPs

Each FastPack® carton contains:

- 50 FastPacks

Each FastPack® IP or FastPack® Contains:

- Paramagnetic Particles, 150 μ L
Paramagnetic particles with covalently coupled testosterone in buffer containing 0.1% sodium azide as a preservative.

- Testosterone Antibody Solution, 100 µL
Antibody solution containing mouse monoclonal antibody labeled with alkaline phosphatase in a protein matrix containing 0.03% Proclin[®] 150 as a preservative.
- Wash Buffer, 2.0 mL
Tris buffer containing surfactants.
- Substrate, 140 µL
ImmuGlow[™]: Indoxyl-3-phosphate and lucigenin in buffer containing preservatives.

Materials required but not provided

- FastPack[®] IP System
- FastPack[®] Testo Calibrator Kit - Cat. No. 25000014

WARNING AND PRECAUTIONS

- For *in vitro* diagnostic use only.
- Do not pipette by mouth.
- Do not eat, drink or smoke in designated work areas.
- Wash hands thoroughly after handling specimen.
- HAMA Interference: some individuals have antibodies to mouse protein (HAMA), which can cause interference in immunoassays that employ antibodies derived from mice⁵.
- FastPack[®] IP and FastPack[®] reagents are stable until the expiration date on the label when stored and handled as directed. Do not use FastPack[®] IP and FastPack[®] reagents beyond expiration date.
- Discard used FastPacks into a biohazard container.
- The components containing Sodium Azide are classified per applicable European Economic Community (EEC) Directives as: Harmful (Xn). The following are appropriate Risk (R) and Safety (S) phrases:
 - R22 Harmful if swallowed.
 - R32 Contact with acids liberates very toxic gas.
 - S2 Keep out of reach of children.
 - S13 Keep away from food, drink and animal feeding stuffs.
 - S36 Wear suitable protective clothing.
 - S46 If swallowed, seek medical advice immediately and show this container or label.

STORAGE INSTRUCTIONS

Store at 2 - 8 °C. Protect from light.

SPECIMEN COLLECTION/PREPARATION

1. Serum samples are required for the FastPack[®] IP and FastPack[®] Testo Immunoassay. Plasma samples should not be used.
2. The National Committee for Clinical Laboratory Standards (NCCLS) provides recommendations for handling, processing, and storing blood.^{3,4}
3. Collect all blood samples observing routine precautions for venipuncture.
4. It is not required that patients fast prior to blood collection.
5. For serum samples:
 - Ensure that complete clot formation has occurred before centrifugation. This takes approximately 30 minutes. Some samples may exhibit increased clotting time, especially those from patients receiving anticoagulant or thrombolytic therapy.
 - Serum should be centrifuged and separated from the clot within 3 hours from time of collection.
 - Remove serum from the cells prior to storage at 2-8 °C.
 - If not tested within 24 hours, the sample should be frozen at -20 °C or colder.
6. Do not freeze samples (-20 °C) for more than two months.
7. Frozen samples should be thawed completely and mixed by gentle inversion prior to use.
8. Samples should be free of Fibrin, Red Blood Cells, or other particulate material for optimal results.
9. Serum samples showing turbidity and/or particulate matter should be centrifuged prior to use.
10. Ensure the samples are free of bubbles.
11. Human samples should be handled in accordance with the OSHA standard on Bloodborne Pathogens.¹⁹

ASSAY PROCEDURE

See FastPack[®] IP System Procedure Manual for information on operating the FastPack[®] IP System.

INSTRUMENTATION

FastPack[®] IP System

DETAILS OF CALIBRATION

During the FastPack[®] IP and FastPack[®] production process, Qualigen generates a master standard curve and places this information in the barcode of each FastPack[®] IP and FastPack[®] label, where it can be read by the FastPack[®] IP Analyzer during the testing sequence. The FastPack[®] IP Analyzer must be calibrated by the user to ensure that it is properly adjusted for the particular lot of FastPacks that is being used. Separate calibrations must be run for each type of test, i.e. Free PSA, Total PSA or Testosterone. The frequency of calibration varies for each test type. For the

FastPack® IP and FastPack® Testo Immunoassay, the FastPack® IP Analyzer must be calibrated once every 14 days or whenever a new lot of Testo FastPacks are to be used.

Whenever the user performs an initial calibration for a particular lot of FastPacks or uses a new lot of calibrator, 2 FastPacks must be run for calibration (duplicates). Whenever recalibration is performed with the same lot of FastPacks and calibrator, 2 FastPacks must be run for calibration. See FastPack® IP System Procedure Manual for “Running a Calibration”.

Use FastPack® Testo Calibrator Kit – Cat. No. 25000014

RESULTS

The FastPack® IP Analyzer uses the information from the barcode to construct a lookup table of x,y values that represent the standard curve and estimates the concentration of unknown samples by linear interpolation.

QUALITY CONTROL

Quality control materials simulate real specimens and are essential for monitoring the system performance of assays. Good Laboratory Practices (GLP) include the use of control specimens to ensure that all reagents and protocols are performing properly. Commercial control products (such as Bio-Rad Liquichek® Immunoassay Plus Control, level 2 and 3) are available which contain testosterone. See FastPack® IP System Procedure Manual for “Running Controls”.

LIMITATION OF PROCEDURE

- Samples can be measured within the reportable range of the analytical sensitivity and the upper end of the calibration range, 16 ng/mL.
- Samples >16 ng/mL should be run using another method. Dilution of out of range results is not recommended.
- Samples from patients who have received preparations of mouse monoclonal antibodies for diagnosis or therapy may contain human anti-mouse antibodies (HAMA). Such samples may show either falsely elevated or depressed values when tested with assay kits employing mouse monoclonal antibodies.
- Heterophilic antibodies in a sample have the potential to cause interference in immunoassay systems. Infrequently, testosterone levels may appear elevated due to heterophilic antibodies present in the patient’s serum or to nonspecific protein binding. If the testosterone level is inconsistent with clinical evidence, additional testosterone testing is suggested to confirm the result.
- For diagnostic purposes, the FastPack® IP and Fastpack® Testo Immunoassay should always be assessed in conjunction with the patient’s medical history, clinical examination and other findings.

EXPECTED RANGE

Serum samples were obtained from 162 random males and 129 random females ranging in age from 18 to 94 years. Samples were obtained from normal, healthy blood donors without any clinically abnormal indications. Testosterone levels were determined using the FastPack® IP and Fastpack® Testo Immunoassay in conjunction with the FastPack® IP Analyzer in order to establish the testosterone concentration in the normal population.

[Testosterone], ng/mL	#Males	#Females
0.0 - 0.5	0	86
0.5 - 1.0	1	16
1.0 - 2.0	12	16
2.0 - 3.0	21	6
3.0 - 10	120	5
>10	8	0
TOTAL	162	129

Note: The expected range reflects the donor population of this study group. Each laboratory should determine their own reference range appropriate for their population.

SPECIFIC PERFORMANCE CHARACTERISTICS

Precision

The reproducibility of the testosterone assay was measured by 2 patient samples in duplicate (n=60, for each sample) over ten days using three analyzers and two lots of reagents. Precision is expressed either as a Standard Deviation (low level), or Coefficient of Variation (high sample). At the lower level the testosterone value is low thus the CV has little meaning.

Sample	Mean (ng/mL)	Between Run	Between Analyzer	Between Reagent Lot	Total
Low Sample	0.14	SD = 0.06	SD = 0.03	SD = 0.01	SD = 0.07
High Sample	7.12	%CV = 10.3	%CV = 3.8	%CV = 0.05	%CV = 10.9

Spike Recovery:

A patient sample was spiked with testosterone from a human based stock material. The stock concentration was 54 ng/mL.

Sample Number	Added Testosterone (ng/mL)	Expected Value Testosterone (ng/mL)	Observed Value Testosterone (ng/mL)	Recovery (%)
1	0	3.98	3.98	100
2	1.0	4.98	5.33	107
3	2.51	6.49	7.14	110
4	5.01	8.99	9.27	103
5	7.52	11.5	10.9	94.8
6	3.51	14.0	13.6	97.1

Dilution Recovery

A high-level testosterone patient sample was diluted with a low level patient sample.

Pool #1	Expected Value Testosterone (ng/mL)	Observed Value Testosterone (ng/mL)	Recovery (%)
Neat	8.38	8.38	—
65%	6.33	6.32	99.8
50%	4.27	5.51	129
25%	2.22	2.93	132

Method Comparison

Clinical samples were used to compare the serum values obtained using the FastPack® IP and FastPack® Testosterone method and the serum value obtained using the DPC Coat-a-Count RIA method. The values were evaluated for agreement using Deming regression analysis. Spearman rank correlation was used to test association between the values.

n	Range of Observation (ng/mL)	Intercept (ng/mL)	Slope	r
144	0.07 - 16.0	-0.004	1.065	0.930

INTERFERING SUBSTANCES

Interfering substances were added to serum samples containing known amounts of testosterone. The value obtained for the serum with each interfering substance was compared to the value obtained for the serum without the interfering substance. These compounds did not show interference at the levels indicated.

Test Compound	Test Concentration
Bilirubin	40 mg/dL
Hemoglobin	1,000 mg/dL
Triglycerides	1,000 mg/dL

Analytical Sensitivity

The analytical sensitivity of testosterone is 0.13 ng testosterone/mL. This value was interpolated from the dose response curve that was minus two standard deviations below the average of the zero calibrator signal (n=14/analyzer/lot).

Analytical Specificity

For the monoclonal antibody used, the following cross-reactivity contained in the table below were found.

Material Tested	Test Levels ng/mL	% Cross-reactivity
5- α -DHT	50	3.16
Androstenedione	1,000	1.15
Androsterone	100,000	0.011
Estradiol	1,000	0.006
Danazol	1,000	0.002
Cortisol	8,000	0.001
DHEA	10,000	0.003
Oxymetholone	100	0.016
Estrone	500	0.003
Corticosterone	5,000	0.003
Methyltestosterone	100	0.030
11-Deoxycortisol	1,000	ND
Progesterone	1,000	0.01
19-Nor Testo	1,000	2.05
19-Nor Testo	500	2.07

ND = not detectable

REFERENCES

- Wilson, JD, George, FW, Griffin, JE: The hormonal control of sexual development. Science, 211: 1278 – 1284, 1981.
- Imperato-Mcginley, J, Guerrero, T, Peterson, RE: Steroid 5 α -reductase deficiency in man: An inherited form of male pseudohermaphroditism. Science, 186:1213 – 1215, 1974.
- Approved Standard – procedures for the collection of diagnostic blood specimens by venipuncture. 5th Edition: H3-A5: 23(32) 2003, National Committee for Clinical Laboratory Standards (NCCLS)
- Approved guideline – procedures for the handling and processing of blood specimens, H18-A2;19(21), 1999. National Committee for Clinical Laboratory Standards (NCCLS).
- Schroff, RJ, Foon, KA, et.al.: Human anti-mouse immunoglobulin responses in patients receiving monoclonal antibody therapy. Cancer Res, 45:879 – 885, 1985
- US Department of Labor, Occupational Safety and Health Administration, 29CFR Part 1910.1030, Occupational Exposure to Bloodborne Pathogens; Final Rule. Federal Register 1991; 56(235): 64175-82.
- Gronowski AM, Landau-Levine M. Reproductive Endocrine Function in Tietz Textbook of Clinical Chemistry. Ed. Edward R. Ashwood, and Carl A. Burtis. 3rd ed. N.p.:W. B. Saunders Company, 1999. 1603-1636.

Qualigen, Inc.
 Carlsbad, CA 92011 USA
 Technical Support
 (760) 918-9165
 (877) 709-2169

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RÉFÉRENCES

- ¹ Wilson, JD, George, FW, Griffin, JE: The hormonal control of sexual development. Science, 211: 1278 – 1284, 1981.
- ² Imperato-Mcginley, J, Guerrero, T, Peterson, RE: Steroid 5 a-reductase deficiency in man: An inherited form of male pseudohermaphroditism. Science, 186:1213 – 1215, 1974.
- ³ Approved Standard – procedures for the collection of diagnostic blood specimens by venipuncture. 5th Edition: H3-A5: 23(32) 2003, National Committee for Clinical Laboratory Standards (NCCLS)
- ⁴ Approved guideline – procedures for the handling and processing of blood specimens, H18-A2;19(21), 1999. National Committee for Clinical Laboratory Standards (NCCLS).
- ⁵ Schroff, RJ, Foon, KA, et.al.: Human anti-mouse immunoglobulin responses in patients receiving monoclonal antibody therapy. Cancer Res, 45:879 – 885, 1985
- ⁶ US Department of Labor, Occupational Safety and Health Administration, 29CFR Part 1910.1030, Occupational Exposure to Bloodborne Pathogens; Final Rule. Federal Register 1991; 56(235): 64175-82.
- ⁷ Gronowski AM, Landau-Levine M. Reproductive Endocrine Function in Tietz Textbook of Clinical Chemistry. Ed. Edward R. Ashwood, and Carl A. Burtis. 3rd ed. N.p.:W. B. Saunders Company, 1999. 1603-1636.

Qualigen, Inc.
Carlsbad, CA 92011, États-Unis
Assistance technique:
(760) 918-9165
(877) 709-2169

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REFERENZEN

- ¹ Wilson, JD, George, FW, Griffin, JE: The hormonal control of sexual development. Science, 211: 1278 – 1284, 1981.
- ² Imperato-Mcginley, J, Guerrero, T, Peterson, RE: Steroid 5 α -reductase deficiency in man: An inherited form of male pseudohermaphroditism. Science, 186:1213 – 1215, 1974.
- ³ Approved Standard – procedures for the collection of diagnostic blood specimens by venipuncture. 5th Edition: H3-A5: 23(32) 2003, National Committee for Clinical Laboratory Standards (NCCLS)
- ⁴ Approved guideline – procedures for the handling and processing of blood specimens, H18-A2;19(21), 1999. National Committee for Clinical Laboratory Standards (NCCLS).
- ⁵ Schroff, RJ, Foon, KA, et.al.: Human anti-mouse immunoglobulin responses in patients receiving monoclonal antibody therapy. Cancer Res, 45:879 – 885, 1985
- ⁶ US Department of Labor, Occupational Safety and Health Administration, 29CFR Part 1910.1030, Occupational Exposure to Bloodborne Pathogens; Final Rule. Federal Register 1991; 56(235): 64175-82.
- ⁷ Gronowski AM, Landau-Levine M. Reproductive Endocrine Function in Tietz Textbook of Clinical Chemistry. Ed. Edward R. Ashwood, and Carl A. Burtis. 3rd ed. N.p.:W. B. Saunders Company, 1999. 1603-1636.

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© 2000 Qualigen, Inc. Tous droits réservés. Qualigen et FastPack sont des marques commerciales ou des marques déposées de Qualigen Inc. Toutes les autres marques commerciales sont la propriété de leurs propriétaires respectifs.